

INSTRUCTIONS FOR USE

VIBRANT SYSTEM

For oral use

Rx ONLY

This "Instructions for Use" contains information on how to use the Vibrant system.


Indications for Use

The Vibrant System is an orally administered Capsule that is indicated for the treatment of adults with chronic idiopathic constipation who have not experienced relief of their bowel symptoms by using laxative therapies at the recommended dosage for at least one month.

1. Vibrant System Components

The Vibrant system is comprised of the following components:


1. Pod
2. USB cable
3. AC/DC USB adapter
4. 2 blister packs of 10 Vibrant Capsules each, in 1 box

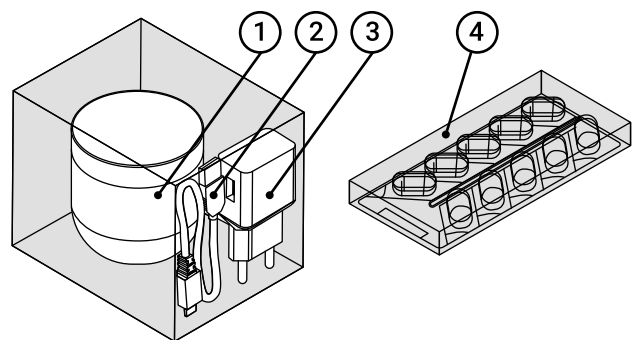

The Pod

The Pod is used to activate the Vibrant Capsule before swallowing the Capsule:

1. Designated Groove for Capsule
2. Light ring indicator
3. Pod lid

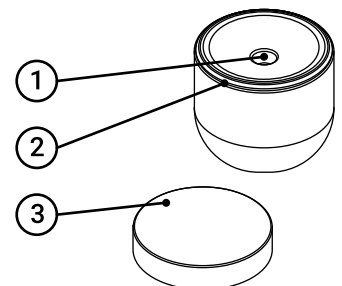

The Capsule should not cause any interference during travel activities such as flying and going through airport security.

2. Important Information for Patients

Contraindications

The Vibrant Capsules are contraindicated for use under the following conditions:

- History of complicated/obstructive diverticular disease
- History of intestinal or colonic obstruction or suspected intestinal obstruction
- Clinical evidence of current and significant gastroparesis
- History of significant gastrointestinal disorder, including any form of inflammatory bowel disease or gastrointestinal malignancy (celiac disease is accepted if the subject has been treated and is in remission), and/or anal fissures and fistulas
- History of Zenker's diverticulum, dysphagia, esophageal stricture, eosinophilic esophagitis or achalasia
- Women who are pregnant or lactating